# Intensiv ProxoStrip

Pat. 699 819

DE

## Produktbeschreibung

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EN

## Product description

- Metal strip, diamond-coated on one side only with two working parts of different grit size: 40µm (red) and 15µm (yellow)
- Perforated anatomical ends for a better gripping
- Height: 2.5mm
- Total length: 80mm
- Thickness: 0.05mm
- Sterilizable and reusable

## Intended use

- Finishing and polishing of fillings and crown borders in interproximal areas.
- Removal of excess material and overhangs in fillings and crown borders.
- Beveling in adhesive restorations in cervical interproximal areas.

## Practical Tips

- Application by dentist only.
- The total length of the strip (80mm) allows an optimal finger support on the adjacent teeth during work.
- To get past the contact area, use the uncoated part of the strip and exert a slight horizontal movement.
- After introduction, work with the less abrasive side (yellow, 15µm), in order to facilitate the access of the more abrasive side (40µm, red). Then begin the finishing procedure using a back-and-forward movement.
- Next, switch again to the less abrasive side (15µm, yellow) to complete the procedure with the polishing.
- Intensiv ProxoStrip must be removed with a vertical movement after repositioning it to the uncoated segment.
- In the event the contact area is very tight, it is useful to start by inserting a wooden wedge to open up a slight separation between the teeth before placing Intensiv ProxoStrip in the area to be treated.

## Maintenance and sterilisation

- Intensiv ProxoStrip is supplied non-sterile. Prior to first use on the patient and immediately after each use, the instruments must be disinfected, cleaned, and sterilized.
- Keep ProxoStrip separate from non-stainless steel instruments, such as rubber polishers and abrasive stones.
- If disinfection by immersion is being used, it is important to use only cleaning/disinfection solutions that provide corrosion protection and strictly observe the concentrations recommended by the manufacturer.
- Sterilization must be performed with validated procedures. If possible, use a single-pulsed or fractionated vacuum autoclave and subvacuum drying.
- Full cleaning, disinfection, sterilization, checking and maintenance instructions can be requested at info@intensiv.ch

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**Intensiv ProxoPolish** (grit size 15µm and 8µm)  
REF PXPO1508/6 Package of 6 Strips

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(EN)

#### Product description

- Metal strip, diamond-coated on one side only with two working parts of different grit size: 40µm (red) and 15µm (yellow)
- Anatomical ends in sterilizable plastic for a better gripping
- Height: 2.5mm
- Total length: 83mm
- Thickness: 0.05mm
- Sterilizable and reusable

#### Intended use

- Finishing and polishing of fillings and crown borders in interproximal areas
- Removal of excess material and overhangs in fillings and crown borders
- Bevellung in adhesive restorations in cervical interproximal areas

#### Practical Tips

- Application by dentist only.
- The color of the grips indicates the grit size of the strip, red 40 µm, yellow 15 µm; the opposite white part indicates the non-diamond coated side of the strip.
- The total length of the strip (83mm) allows an optimal finger support on the adjacent teeth during work.
- To get past the contact area, use the uncoated part of the strip and exert a slight horizontal movement.
- After introduction, work with the less abrasive side (yellow, 15µm), in order to facilitate the access of the more abrasive side (40µm, red). Then begin the finishing procedure using a back-and-forward movement.
- Next, switch again to the less abrasive side (15µm, yellow) to complete the procedure with the polishing.
- Intensiv ProxoStrip Plus must be removed with a vertical movement after repositioning it to the uncoated segment.
- In the event the contact area is very tight, it is useful to start by inserting a wooden wedge to open up a slight separation between the teeth before placing Intensiv ProxoStrip Plus in the area to be treated.

#### Maintenance and sterilisation

- Intensiv ProxoStrip Plus is supplied non-sterile. Prior to first use on the patient and immediately after each use, the instruments must be disinfected, cleaned, and sterilized.
- Keep Intensiv ProxoStrip Plus separate from stainless steel instruments.
- If disinfection by immersion is being used, it is important to use only cleaning/disinfection solutions that provide corrosion protection and strictly observe the concentrations recommended by the manufacturer.
- Sterilization must be performed with validated procedures. If possible, use a single-pulsed or fractionated vacuum autoclave and subvacuum drying.
- Full cleaning, disinfection, sterilization, checking and maintenance instructions can be requested at info@intensiv.ch

In the unlikely case of an incident caused by the device please refer to the manufacturer and competent authorities.

Intensiv SA declines any liability for any use of the instrument other than that specified in these instructions for use.

**Intensiv ProxoStrip Plus** (grit size 40µm and 15µm)  
REF PXP4015/6 Package of 6 Strips  
REF PXP4015/12 Package of 12 Strips

#### Additional models

**Intensiv ProxoStrip Anterior** (grit size 40µm and 15µm)  
REF PXA4015/6 Package of 6 Strips  
Ref. PXA4015/12 Package of 12 Strips

**Intensiv ProxoStrip** (grit size 40µm and 15µm)  
REF PX4015/6 Package of 6 Strips  
REF PX4015/12 Package of 12 Strips

**Intensiv ProxoContour** (grit size 60µm and 40µm)  
REF PXC6040/6 Package of 6 Strips

**Intensiv ProxoContour Coarse** (grit size 80µm and 40µm)  
REF PXCC8040/6 Package of 6 Strips

**Intensiv ProxoPolish** (grit size 15µm and 8µm)  
REF PXP01508/6 Package of 6 Strips

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